LM NÁSTROJE

LM NÁSTROJE

O firmě LM-Instruments, Oy 4

Diagnostika 11

Periodoncie 17

Léčení 31

Endodoncie 45

Ortodoncie 51

Sety nástrojů 54

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Obsah

LM NÁSTROJE

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- Hand files and reamers
- Root Canal Explorer
- Root Canal Excavators
- Endo Plugger
- Endodontic Surger

LM NÁSTROJE

LM-EndoMax

LM endodontické 's endodontic hand fi les with large handles are supreme in ergonomics.

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Soft and thick silicone handle improves the grip and makes fi ling and rotating more comfortable. LM-EndoMax fi les and reamers adhere to the contour of the canal, cleaning all surfaces evenly and effectively.

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- kulatá silikonová rukojeť

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- the blade can easily be maneuvered on the various surfaces of the tooth

- the blade is deeply set, with the shaft preventing the blade from damaging soft tissue and allowing for the use of great

tractive and lateral force when carving out excess material

LM NÁSTROJE

LM ErgoHold 3

LM- ErgoHold 3 LM 9003

A new type of surgical handle

- excellent grip
 - precise blade control
 - controlled rotation
 - blade position responsiveness maintained
 - controlled movement through tissue
 - no. 3 fi tment
 - elastic silicone coating
- PAT.PEND.

CompoKnife works well
in approximal fi nishings.

Ideal for fi nishing of

vacuum tray edges used
in bleaching treatments.

Outstanding instrument for
the removal of thin layer of
bonding resin.

LM MultiSeptor

Effective and easy-to-use saliva ejector

LM-MultiSeptor keeps the area being
treated dry – both lingual and buccal. It
functions equally well on both the upper
and lower jaw.

- can easily be formed to the desired shape
- comfortable for the patient
- leaves ample space for procedures
- an invaluable aid when working
unassisted.

The MultiSeptor is used in e.g. in
restorative and endodontic procedures
as well as in orthodontics and crown and
bridge work. The MultiSeptor is compatible
with all standard suction units.

Care: disinfect, rinse, remove connections
clean, and sterilize.

Connections must be cold-sterilized.

LM- MultiSeptor, 40 cm LM 2020

LM- MultiSeptor, 50 cm LM 2020A

Spare Parts

Replacement tubes = 5 tubes (40 cm) + spring
LM 2021

Replacement tubes = 5 tubes (50 cm) + spring
LM 2021A

LM NÁSTROJE

BRUSH APPLICATION TIPS

Brush Application Tips are for one time use only!

- used for applying liquid treatment agents to all dental surfaces or cavities
 - excellent for placing cavity liners, bonding and priming agents and etchants
 - by bending the shank of the application tip the access to into the cavities can be optimised
- Application tips are available in three different colours (red, blue, yellow) to ease the identification and to coordinate multi-step procedures.

Brush Application Tip, yellow

50 pieces

LM 758Y

Brush Application Tip, blue

50 pieces

LM 758B

Brush Application Tip, red

50 pieces

LM 758R

Use Brush Application Tips together with LM-BrushHolder (LM 481-758).

See page 38 for more information on the BrushHolder.

CODE RINGS

- LM 5000 blue
- LM 5001 green
- LM 5002 orange
- LM 5003 red
- LM 5004 yellow
- LM 5005 black
- LM 5006 white
- LM 5007 brown
- LM 5008 lavender
- package 50 pcs/

LM SERVO KAZETY

Save time, money and personnel

- made of an exceptionally durable special plastic
- cassettes tolerate all common detergents, washers, and methods
- max. 180°C / 355°F
- standard dimensions – can be kept in all commonly used instrument cabinets, instrument maintenance equipment, and covered trays
- one-piece, light-weight

LM- Servo 5

LM 6650

- for five instruments
 - dimensions: 180 x 86 x 28 mm
- PAT.

LM- Servo 8

LM 6680

- for eight instruments

LM NÁSTROJE

- dimensions: 180 x 136 x 28 mm
PAT.

CODE BUTTONS

On both of the cassette's long sides there are holes for three code buttons

- buttons sold separately
- package 20 pcs/colour

LM 6000 blue
LM 6001 green
LM 6002 orange
LM 6003 red
LM 6004 yellow
LM 6005 brown

LM PROCOLOR INSTRUMENT DISINFECTANT

Effective protection

LM-ProColor is a new aldehyde-free disinfectant for hand instruments that combines a wide effective spectrum, high cleaning power and composition that protects various materials.

LM-ProColor is also suitable for use in ultrasonic cleaners.

Safe and long-lasting

LM-ProColor is a long-lasting concentrate, whose pleasant aroma and colour enhance user comfort. One two-liter bottle produces 100 liters of cleaning solution.

Composition: 100 g LM-ProColor contains: 9 g alkyl-benzyl-dimethylammonium chloride, 0.1 g guanidine compound, non-ionic surfactants, alkaline detergents, corrosion inhibitors, adjuvants in aqueous solution.

Effective spectrum: bactericidal inclusive of TB bacteria, fungicidal, virucidal (enveloped viruses, HBV₁, HCV₂, HIV₃).

- 1) by electron microscopy
- 2) Test for BVDV surrogate virus
- 3) Effectiveness against HBV implies effectiveness against HIV.

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Ultrasonic cleaner: use instrument cassettes to prevent damage

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RONTO LM PLUS BRUSNÝ KÁMEN

Instruments used in treating hard tissue blunt quickly, naturally compromising efficiency and results. Periodontal instruments should be sharpened after every use. As sharpening by hand takes time and wears down instruments, choose the better alternative!

Easy and quick

- simple method - the blade is held in the correct sharpening position and the grinding stone rotates
- sharpening one instrument only takes a few seconds
- interactive demonstration and instruction CD, and handy quick guide

Versatile

- for both periodontal and preparation instruments
- for both right or left-handed
- the direction of rotation can be selected

Accurate and safe

- a dual-position instrument rest. The upper position is for preparation instruments and the lower for periodontal instruments

LM NÁSTROJE

- the original blade form is preserved
- when sharpening round or arched blades, the cover is turned simultaneously resulting in the correct blade angle and an even, sharp cutting edge
- low rotational speed - the blade will not heat up nor will its temper be affected

The fine-grain ceramic stone is 400 grit. Instrument wear is reduced as much as possible, thus increasing its service life. Finer and coarser sharpening stones are available as spare parts.

LM- RondoPlus

LM 8600

Package includes: sharpening machine, foot pedal, 400 grit sharpening stone, 2 LM-Fingos, 5 emery tapes, instruction CD and quick guide.

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LM-ErgoGrip, LM-ErgoGrip Light a LMErgoGrip Focus Light.

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wrenches, 1 torque tip wrench, 1 of each silicone knob and button, 1 foot pedal.

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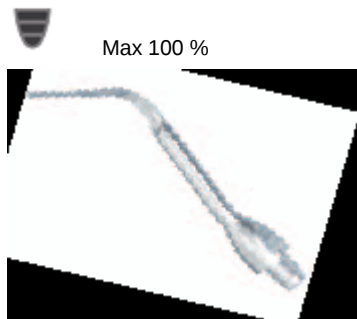
Scaling

Scaling is precision work. Proper ultrasonic instrument selection is of primary importance in achieving complete periodontal debridement. Our wide variety of tips gives you the freedom to choose.

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Working mode coding

PE-31

LM 100131

Used for lingual, buccal and approximal supragingival scaling.

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PE-37L

LM 100137L

Universal tip for subgingival scaling, furcations, supragingival fi ne scaling and spot removal. Tip angled to left for better access to furcations.

PE-37R

LM 100137R

Universal tip for subgingival scaling, furcations, supragingival fi ne scaling and spot removal. Tip angled to right for better access to furcations.

PERIODONCIE

PE-33

LM NÁSTROJE

LM 100133

Used for subgingival scaling and furcations.

PE-33L

LM 100133L

Used for subgingival scaling and furcations. Tip angled to left to left for better access to furcations.

PE-33R

LM 100133R

Used for subgingival scaling and furcations. Tip angled to right for better access to furcations.

PE-38

LM 100138

Used for lingual and buccal subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal.

PE-38L

LM 100138L

Used for subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal. Tip angled to left for better access to furcations.

PE-38R

LM 100138R

Used for subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal. Tip angled to right for better access to furcations.

PE-40

LM 100140

Used for deeper lingual and buccal subgingival scaling. Also suitable for supragingival fine scaling and spot removal.

PE-40L

LM 100140L

Used for deeper subgingival scaling and molar furcations. Tip angled to left for better access to furcations.

PE-41P

LM 100141P

Used for universal, lingual and buccal supragingival scaling. Can also be used for shallow pockets.

PE-41AF

LM 100141AF

Used for lingual, buccal and approximal supragingival scaling. Also for the removal of overhangs.

PE-41PF

LM 100141PF

Used for lingual, buccal and approximal supragingival scaling.

PE-40R

LM 100140R

LM NÁSTROJE

Used for deeper subgingival scaling and molar furcations. Tip angled to right for better access to furcations.

PE-40H

LM 100140H

Used for deeper subgingival scaling. Especially suitable for lingual scaling of incisors.

PE-41PS

LM 100141PS

Used for deeper lingual and buccal subgingival scaling. Also suitable for supragingival fine scaling and spot removal.

IMPLANT MAINTENANCE

IM-1

LM 100301 Pack of 4

Used on implants for the removal of calculus on fixtures, abutments and crowns. The working end is coated with PEEK® material to avoid damaging or discolouring the implant. Used together with the instrument holders IH-1 or IH-2.

IM-2

LM 100302 Pack of 4

Same design as IM-1, but the working end is angulated for better access to difficult-to-reach surfaces. Used together with the instrument holders IH-1 or IH-2.

Minimally invasive excavation

With the aid of diamond-coated ultrasonic tips small preparations can be carried out without the unnecessary removal of the surrounding tooth substance. The tips can also easily be used for crown margins and cleaning fissures. The small diameter of the tips make them very suitable for treating children and the low noise level in comparison with rotating instruments is especially appreciated by nervous patients.

MI-1

LM 100601

Straight tip, used primarily for incisors and more accessible surfaces.

MI-2

LM 100602

Slightly contra-angled, used for lingual and buccal work on molars.

MI-3

LM 100603

Contra-angled, used on lingual, buccal and distal areas.

MI-4

LM 100604

Heavily contra-angled, used on lingual, buccal and distal areas on molars and furcations.

Endodontics

Ultrasonic tips can be utilised in many areas in endodontics. They are truly excellent in the removal of posts, the removal of dentin in pulp chambers, finding and widening orifices, preparing canals, removing broken instruments and cleaning prepared canals.

Pulp chamber

EN-1

LM NÁSTROJE

LM 100201

Ball shaped. Used for the removal of crowns, bridges and a variety of posts set with various cementing agents.

The tip is placed against the post and then moved in a circular fashion to remove it. Length 17 mm.

EN-2

LM 100202

Flat and round diamond-coated surface on the tip. Used for planing attached pulp stones from the pulp chamber floor. Compared to rotating instruments the risk of perforating the tooth is small. Length 17 mm.

EN-3

LM 100203

All-purpose, conical and diamondcoated tip primarily used in the pulp chamber for removing pulp stones, dentin and old fillings. Also for finding hidden openings. Length 17 mm.

EN-4

LM 100204

Round diamond-coated tip for removing gross dentin, for moving access line angles, for cutting a groove in the mesial access wall to drop into the second mesial-buccal canal (MB2) systems, and for quickly and carefully unroofing pulp chambers. Length 16 mm.

EN-12

LM 100212

Flat and diamond-coated tip. Used in the pulp chamber for removing pulp stones and also for removal of dentin and old fillings. Compared to rotating instruments the risk of perforating the tooth is small.

g end is angulated
ch surfaces. Used
rs IH-1 or IH-2.
of calculus on
he working end
id damaging
her with

Canal preparation

EN-6

LM 100206

Primarily used in the coronal area of the root canal to trephine around posts. Also for chasing calcified canals halfway up a root. The water supply is placed at the furthest extremity of the tip for increased rinsing and cooling of the treated area. Length 19 mm.

EN-5

LM 100205

Used in the coronal and apical part of root canals. Amongst other things the tip can be used to remove posts, widen calcified canals, remove hard fillings, and remove broken instruments and other intra-canal obstructions. Length

LM NÁSTROJE

17 mm.

EN-7

LM 100207

Same shape as tip EN-5, but 2 mm longer, and same areas of use. Used in the coronal, middle and apical one-thirds of roots. Length 19 mm.

EN-8

LM 100208

Same shape as tip EN-5, but 7 mm longer, and same areas of use. Used in the coronal, middle and apical one-thirds of roots. Length 24 mm.

EN-14

LM 100214

Used in combination with the instrument holders IH-1, IH-2, IHS-1 or IHS-2. Used as a plugger for lateral condensation of guttapercha. Length 25 mm.

EN-13

LM 100213

Diamond coated tip used in combination with the instrument holders IH-1, IH-2, IHS-1 or IHS-2. Used in the coronal, middle and apical part of root canals. Used to remove posts, widen calcified canals, remove hard fillings, remove broken instruments and other intracanal obstructions. Length 17 mm.

EN-15

LM 100215

Used in combination with the instrument holders IH-1, IH-2, IHS-1 or IHS-2. By using EN-15 in a fluid-filled root canal the ultrasonic vibrations clean the canal very efficiently. The resulting cavitation effect gives a proven high degree of cleanliness. Length 25 mm.

Removal of broken instruments

Tips **EN-9**, **EN-10** and **EN-11** are made of titanium and have no diamond coating. They hold a bend if applied forcefully. Titanium tips cut evenly and have an exceptional tactile sense. The small diameter makes them suitable for use in the apical part of root canals. Primary area of use is to isolate and remove broken instruments. Even broken instruments that are firmly stuck in the apical part of the root can often be removed by ultrasonic vibrations. Used in the mid and apical part of a root with illumination and magnification.

EN-9

LM 100209

Length 20 mm.

EN-10

LM 100210

Length 24 mm.

EN-11

LM 100211

Length 27 mm

Caution! All endodontic tips are extremely sensitive and if not used correctly or with too much power or force they are subject to breakage. Do not exceed

LM NÁSTROJE

*recommended working mode
and always place the tip on the
tooth surface before turning on
the power*

File and instrument holders

FH-1, FH-2 and FH-3 are holders for hand files. By using the hand file in a fluid-filled root canal the ultrasonic vibrations clean the canal very efficiently. The resulting cavitation effect gives a proven high degree of cleanliness.

FH-1

LM 100401

70° angle.

FH-2

LM 100402

90° angle.

FH-3

LM 100403.

110° angle.

Max 40 %

Working mode coding Endo function Max 60 % Max 70 % Max 100 %

File and instrument holders

IH-1

LM 100404

Holder for implant maintenance tips and AP-1 and AP-2 tips for apical surgery. Can also be used with endosonic files and other instruments with a diameter of 0.8 mm. 125° angle.

Working mode coding: See chosen instrument

IH-2

LM 100405

IH-2 has the same shape and area of use as IH-1. 100° angle.

Working mode coding: See chosen instrument

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Koncovky potažené diamantem jsou určeny pro otevření a čištění kořene při apikální chirurgii. The diamond-coated tips are used for opening and clearing the root when doing apical surgery. The long necks of the tips give excellent visibility and the ultrasonic vibrations in combination with the diamond coating give a higher degree of tactility and precision compared to rotating instruments.

AP-1

LM 100507

Used in combination with the instrument holders IH-1, IH-2, IHS-1 or IHS-2. With the aid of the instrument holder the AP-1 can be turned to precisely the angle needed for the treatment. Available in a pack of four. Diameter 0.3 – 0.8 mm. 110° angle.

AP-2

LM 100508

Same area of use as AP-1. AP-2 has a slimmer design and is therefore better suited for small roots. Available in a pack of four.

Diameter 0.5 mm. 100° angle.

AP-3

LM 100501

AP-3 can be used as a universal tip on all teeth. The outermost 3 mm of the tip is diamond-coated to enable effective removal of tooth substance.

Diameter 0.5 mm. 80° angle

LM NÁSTROJE

AP-4

LM 100502

Same shape and area of use as AP-3, but is 0,7 mm in diameter making it more suitable for larger roots.
80° angle.

AP-5

LM 100503

Designed for use on molar roots in the maxillary right and mandibular left.
Diameter 0.5 mm. 75° angle.

AP-6

LM 100504

Designed for use on molar roots in the maxillary right and mandibular left.
Diameter 0.5 mm. 110° angle

AP-7

LM 100505

Designed for use on molar roots in the maxillary left and mandibular right.
Diameter 0.5 mm. 75° angle.

AP-8

LM 100506

Designed for use on the roots of molars in the maxillary right and mandibular left.
Diameter 0.5 mm. 110° angle.

Caution! All apical surgery tips are extremely sensitive and if not used correctly or with too much power or force they are subject to breakage. Do not exceed recommended working mode and always place the tip on the tooth surface before turning on the power.

Apical surgery tips 3-8 are also available as sterile versions. See page 73 more information!

LM NÁSTROJE

Universal

PE-37R LM 100137R

Universal tip for subgingival scaling, furcations, supragingival fine scaling and spot removal. Tip angled to right for better access to furcations.

PE-41PF LM 100141PF

Used for lingual, buccal and approximal supragingival scaling.

PE-41P LM 100141P

Used for universal, lingual and buccal supragingival scaling. Can also be used for shallow pockets.

PE-41AF LM 100141AF

Used for lingual, buccal and approximal supragingival scaling. Also for the removal of overhangs.

Periodontics

PE-33 LM 100133

Used for subgingival scaling and furcations.

PE-33L LM 100133L

Used for subgingival scaling and furcations. Tip angled to left to left for better access to furcations.

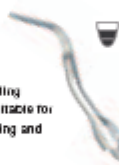
PE-33R LM 100133R

Used for subgingival scaling and furcations. Tip angled to right for better access to furcations.

PE-38 LM 100138

Used for lingual and buccal subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal.

PE-38L LM 100138L

Used for subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal. Tip angled to left for better access to furcations.

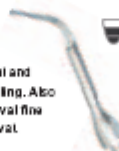
PE-38R LM 100138R

Used for subgingival scaling and furcations. Also suitable for supragingival fine scaling and spot removal. Tip angled to right for better access to furcations.

PE-40 LM 100140

Used for deeper lingual and buccal subgingival scaling. Also suitable for supragingival fine scaling and spot removal.

PE-40L LM 100140L

Used for deeper subgingival scaling and molar furcations. Tip angled to left for better access to furcations.

PE-40R LM 100140R

Used for deeper subgingival scaling and molar furcations. Tip angled to right for better access to furcations.

PE-40H LM 100140H

Used for deeper subgingival scaling. Especially suitable for lingual scaling or incisors.

PE-41PS LM 100141PS

Used for deeper lingual and buccal subgingival scaling. Also suitable for supragingival fine scaling and spot removal.

LM NÁSTROJE